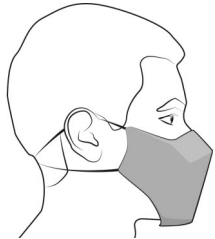
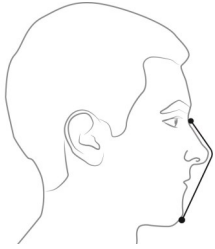
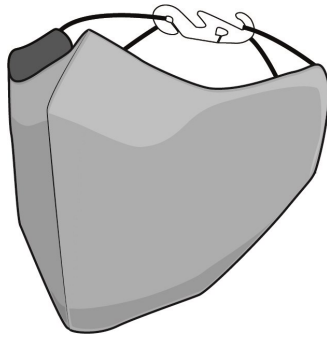
Protective Mask
Anti-projection type

Included in packaging:
1 Facembrace® Protective Mask
1 plastic extension hook
1 User Instructions

SIZE SELECTION (TAILLE)

EN- Measure from the hollow of the nose between the eyes to the tip of the chin.

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SMALL (PETIT)	110mm — 125mm
MEDIUM (MOYEN)	126mm — 139mm
LARGE (GRAND)	140mm — 155mm

GOOD MASK FIT

A well-fitted Facembrace® Protective Mask should cover the chin and the nose halfway up, and rest almost everywhere around the face, without tightening. Leave a ventilation opening under the chin.

It is not a N95 respiratory protection mask nor an ASTM certified product.

PRODUCT DESCRIPTION

This Facembrace® Protective Mask is an anti-projection mask type. It traps the droplets the wearer expels or prevents those from infected sources from reaching its nose and mouth's mucosa.

- Offered in three sizes for a better fit;
- Washes easily;
- Dries very quickly;
- Recovers its original state after washing.

It is not a N95 respiratory protection mask nor an ASTM certified product.

GUARANTEE AND RESTRICTIONS

This Facembrace® Protective Mask is guaranteed for 30 days from the date shown on the purchase receipt, against any manufacturing defect noted before its first use. It is therefore very important to examine your mask as soon as the package is opened. Rheabrio™ Customer Service must confirm that the guarantee applies. Under certain conditions, Rheabrio™ will reimburse the defective product.

The Guarantee does not cover :

- A product modified by the buyer;
- Scratches, tears and discoloration of the product;
- Damages resulting from an accident, neglect, misuse or abuse, installation or storage;
- Damages caused by unrecommended cleaning products;
- Damages resulting from insufficient or inadequate maintenance, or from use other than that for which the product was designed.

CAUTION: The mask may present risks of suffocation for babies and children under 2 years of age, people with respiratory problems and people who are unconscious, incapable or unable to remove it without help.

RHEABRIO™ EXPRESSLY DISCLAIMS ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, CONTRACTUAL OR LEGAL, INCLUDING ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

NORMAL CONDITIONS OF USE

The safe use of Facembrace® products is your sole responsibility.

- Store the never used protective mask inside its original packaging in a clean, dry and uncontaminated place;
- Clean the protective mask between each use;
- Dispose of your protective mask according the standards and laws in force in your region;
- Do not place in the microwave or warm before wearing;
- Do not wear when damp;
- At the end of its useful life, throw away your protective mask according to the standards and laws in force in your region.

HOW TO INSTAL THE MASK

A well-fitted Facembrace® Protective Mask should cover the chin and the nose halfway up, and rest almost everywhere around the face, without tightening. Follow these few steps carefully to maximize your comfort and protection.

1. Wash your hands carefully and take your glasses off if you wear them.
2. Take your Facembrace Protective Mask out of its packaging and hold it by its elastic bands.
3. Put the mask on your face, in a way that it covers your nose and chin (Fig. 1).
4. Connect the elastics behind your neck with the hook provided (Fig. 3).
5. Three attachment positions for the elastic allow better adjustment (Fig. 4).
6. Lightly pinch the protective mask to adjust it to your nose.
7. Place the mask in the middle of the nose so that there is a ventilation opening under the chin. (Fig. 2)
8. Adjust the position of the protective mask on your face if necessary, by holding it by the sides.

Fig.1

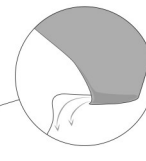
Fig.2

Fig.3

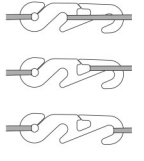
Fig.4

At all times, when using the protective mask, avoid touching the outside of the mask with your hands as it may be contaminated.

HOW TO REMOVE THE MASK

1. Remove your mask by detaching the elastic bands from the hook, **without touching the front of the mask with your hands.**
2. Place it alone in a closed temporary container until it is ready to wash.
3. Wash your hands carefully after handling.

HOW TO CLEAN THE MASK

1. Soak the mask in warm soapy water; the same temperature as for washing your hands.
2. Using a mild hand soap, preferably without perfume or color, wash all parts of the protective mask with your fingers. **Do not rub or brush the fabric of the mask and do not add other cleaning product.**
3. Rinse the mask with clear water to remove the soap and pat it dry in a clean towel.
4. Hang the protective mask by one of its elastic band and allow it to air dry in a clean place.

The Rheabrio™ team is proud to offer you this quality product, well-designed to ensure your well-being and protect your health.

RHEABRIO™

Head Office: 9625-D, Ignace Street, Brossard, QC J4Y 2P3, CANADA

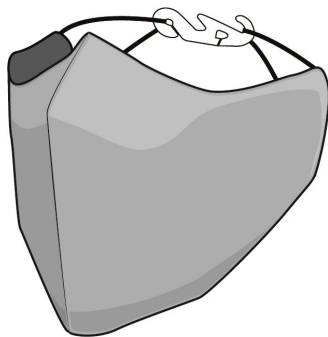
For technical assistance, or for information about our products or about this guarantee, you can contact Rheabrio™ Customer Service.

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